













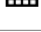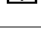


DigitalScope Technical Requirements

DigitalScope™ is a web-based whole slide image (WSI) retrieval and viewing system. It is best viewed with the latest versions of Edge, Chrome, Firefox or Safari.

DigitalScope Viewer Tool Functions (<i>Tools in italics are displayed if available</i>)	
Icon	Tool
	Navigation - Move the image and change the magnification
	FullScreen - Toggle on and off the full screen function
	Panning - The default setting has a motion somewhat like Google Earth and does not stop when you stop the cursor. Click this to change to smooth panning
	<i>Case History</i> - View the case history if available
	<i>Discussion</i> - View the case discussion if available
	<i>Regions of Interest</i> - View the preselected regions of interest if present
	Image Adjustments - Adjust the brightness and contrast
	Overlays - Insert gradient overlays for cell counts
	Tracker -View thumbnail highlighting the current and previously viewed regions
	Measuring Tool - Measure in microns distances between selected points
	Link Tool -Generate links to the image with selected attributes
	Annotations - Allow administrators to annotate images, not visible with routine viewing.
	Student Annotations - Allow students to annotate images, do a screen capture of the area selected, and save or send the image.
	Cell Differential - Allow the viewer to perform a differential on a peripheral smear or a bone marrow
	Help - Display additional information

Navigation Toolbar Functions	
Icon	Tool
	Home - Return to the 0x magnification
	Panning - Move the image in the chosen direction The image can also be moved using your mouse to click-and-drag
	Magnification Slider - Adjust the magnification The magnification can also be adjusted by scrolling the mouse wheel
	Magnification Buttons - Adjust to a selected magnification








Regions of Interest Toolbar Functions	
Icon	Tool
	Region of Interest Information - Display information for specific regions of interest

Image Adjustments Toolbar Functions	
Icon	Tool
	Brightness - Adjust the brightness of the image
	Contrast - Adjust the contrast of the image
	Reset - Reset the brightness and contrast

Overlays Toolbar Functions	
Icon	Tool
	Miller Disc - Overlays a Miller graticule for cell counts
	Grid - Overlay a grid
	Color Dropdown - Change the color of the overlay